



CD Medical

TECHNICAL DATA SHEET - EMOLLIN

Product:	Emollin Emollient Spray
Description:	A topical multi-dose aerosol containing Liquid and White Soft Paraffin B.P. in a volatile silicone base. The aerosol is pressurized with Nitrogen.
Category:	Emollient.
Size:	50ml, 150ml and 240ml aerosol cans
Class:	Class I Medical Device
Regulatory status:	Device has UKCA and CE marks.
Drug Tariff:	240mL Available on NHS Drug Tariff (Part IXA Approved)
Indications:	Protection of skin from minor irritation and soreness. Emollin created a barrier that protects irritated and intact skin from damage. Emollin is preservative free and can be applied by a 'no touch' technique for very sensitive skin. The barrier effect soothes and protects chapped and cracked skin from further damage.
Contraindications:	Sensitivity to any of the ingredients.
Warnings/Precautions:	For external use only. Take care is any of the product is accidentally sprayed on to the floor or other hard surface – Hazard of slipping. Avoid direct contact between the spray nozzle and the surface of the skin. Keep out of reach of children. Do not spray near or into the eyes.
Directions for use:	Spray area to be treated from a distance of about 10 to 20cms, applying a thin unbroken layer. Can be used held at any angle. Replace cap after use.
Storage/Disposal:	Pressurised container, do not expose to temperatures above 50degC. Protect from direct sunlight. Do not puncture or bum even when empty. Observe local aerosol disposal regulations and requirements. Keep out of reach of children.

Further information available on web site: www.cdmedical.co.uk

Issue Date: June 2022

Doc No.			
Version. No.	02	Pages	1 of 1