



CD Medical

TECHNICAL DATA SHEET - IRRIPOD

Product:	Sterile topical saline solution.
Description:	A polythene ampoule (pod) containing 0.9% Sodium Chloride Ph. Eur. odourless, clear sterile aqueous solution.
Category:	Wound irrigation fluid.
Size:	20ml.
Class:	Class IIb Medical Device
Regulatory status:	Device has UKCA and CE marks.
Drug Tariff:	Available on NHS Drug Tariff
Indications:	For the topical irrigation and cleansing of wounds.
Contraindications:	Do not mix with other fluid except as directed.
Warnings/Precautions:	For external use only. Do not use for injections. Avoid direct contact between the spray nozzle and the surface of the wound. Keep out of reach of children. Do not use if the product is not clear. Do not use if the product has leaked before opening. For use on one occasion only – discard any leftover product. Not to be used with other irrigation fluids containing silver, lead or mercurials.
Directions for use:	Snap open the pod by twisting off the seal to reveal the nozzle. Hold firmly and squeeze to apply the solution directly to the area to be irrigated, or invert and squeeze gently to deliver the product dropwise.
Storage/Disposal:	The product should be stored at a temperature between 5 – 25°C, in a clean area well out of the reach of children. Empty pods can be disposed as general waste or recycled

Further information available on web site: www.cdmedical.co.uk

Issue Date: June 2022

Doc No.			
Version. No.	02	Pages	1 of 1