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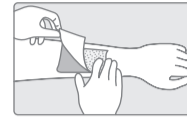
CEMAG Care - 55, rue de Turbigo - 75003 Paris – France  
www.cinesteamcare.com

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**DESCRIPTION**

Cinesteam® is a non-adhesive secondary dressing designed to absorb exudates and eliminate unpleasant odours. It consists of an absorbent layer together with a sachet containing cinnamon.

Composition: polyethylene, polyester, polypropylene and cinnamon.

Cinnamon adsorbs unpleasant-smelling volatile compounds emanating from the wound and masks residual odours with spice natural fragrance.

**INDICATIONS**

Cinesteam® is a secondary dressing indicated for the management of malodorous wounds.

**CONTRAINDICATIONS**

Cinesteam® should not be used on patients with known sensitivity to any of its components, in particular cinnamon.

**DIRECTIONS**

Cinesteam® should be applied on top of an appropriate primary dressing (it is not intended to be in direct contact with the wound). The absorbent layer (white face) should be applied facing the wound, over the primary dressing.

**Application of Cinesteam®**

1. Select the size of Cinesteam® that covers the primary dressing. If necessary, more than one dressing Cinesteam® may be used.
2. Remove Cinesteam® from its primary packaging and apply it directly onto the primary dressing.
3. Keep Cinesteam® in place with any suitable medical material.

**Changing Cinesteam®**

Gently peel off Cinesteam®.

Cinesteam® can be left in place as long as its absorption and odour masking capacities are satisfactory. As a precaution, it is recommended not to leave it in place more than 24 hours.

**PRECAUTIONS**

Do not use Cinesteam® as a primary dressing.

Do not cut Cinesteam® because this could cause cinnamon leakage.

Cinesteam® should not be used if its primary packaging is open or damaged or if the dressing itself is torn.

Do not use Cinesteam® together with an active charcoal dressing, which may impair its odour masking ability.

Do not cover Cinesteam® with an occlusive fastening material or dressing.

Do not eat the cinnamon of Cinesteam®.

Do not bathe or shower when wearing Cinesteam®.

Do not immerse Cinesteam® in water.

Keep out of the reach of children.

Report any serious incidents which occur in connection with Cinesteam® to Cemag Care and to the healthcare competent authority of your country.

If you have any further questions, ask your doctor, pharmacist or nurse.

**AVAILABLE SIZES**

Cinesteam® is available in the following sizes:

- 11 x 11 cm
- 11 x 19 cm

**STORAGE AND SPECIFIC INSTRUCTIONS**

Ship and store flat.

Keep away from sunlight.

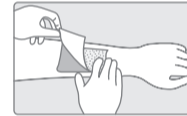
Keep dry.

Do not re-use.

Apply the white face towards the wound.

Do not cut the dressing.

Read the package leaflet before use.

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